



Bassi Hand Plugger Kit™ (4 pcs)

NiTi Hand Plugger System

RX Only. For Dental Use Only.

Composition

Bassi Hand Plugger Kit™ contains 4 dual tipped sizes of nickel titanium alloy: .35mm and .70mm tips (green); .40mm and .80mm tips (black); .50mm and 1.0mm tips (yellow); .60mm and 1.2mm tips (blue).

Indications for Use

Suitable for use in all thermoplastic obturation techniques.

Warnings

Do not heat the tip with fire, do not drop it on the floor, must be sterilized before use.

Contraindications & Adverse Reactions

None Known.

Precautions

1. Before clinical use, consult the IFU.

Bassi Hand Plugger Kit™ Clinical IFU

1. The length of the canal should be divided into three parts (cervical, middle and apical).
2. Select one Bassi Hand Plugger™ (diameter) for depth. Use silicone stops to set the limit.
3. The Bassi Hand Plugger™ chosen should be slightly loose inside the root canal within established limits.
4. The correct choice of Hand Plugger will prevent the clinician from damaging the dental walls. The Bassi Hand Plugger™ should compact only the Gutta Percha cone.