



Bassi Heat Plugger™ (1 pc)

Stainless Steel Single Tip

RX Only. For Dental Use Only.

Composition

The Bassi Heat Plugger™ contains one single tip and the functional portion of the instrument is stainless steel.

Indications for Use

Used in thermoplastic techniques and for cutting Gutta Percha.

Warnings

The tip must only be heated inside the root canal to avoid burning. Must be sterilized before use.

Contraindications & Adverse Reactions

None Known.

Precautions

1. Before clinical use, consult the IFU.

Bassi Heat Plugger™ Clinical IFU

The Bassi Heat Plugger™ is used to heat and cut Gutta Percha automatically.

1. Connect the Bassi Heat Plugger™ to the pen of the “touch ‘n’ heat” device or similar.
2. Attach the tip through the fastening device.
3. Use a silicone stop to set the maximum point within the root canal that the tip should reach.
4. Heat the tip to heat and compact the Gutta Percha according to the technique chosen by the clinician.
5. There is no need for too much pressure to perform the hot condensation. Too much pressure can damage the Bassi Heat Plugger™.
6. Do not use the Bassi Heat Plugger™ for more than 3 seconds inside the root canal to avoid dentin heating.